

Keeler BIO Face Shield Installation Instructions

Ref: AX17448



Step One

To install the Keeler BIO face shield, unsnap the front of the headband foam liner.



Step Two

Once the snaps are exposed, align the holes of the face shield with the headband snaps.



Step Three

After the holes are aligned, snap the headband foam liner into the original closed position.



For more information please call 610.353.4350 or email customerservice@keelerusa.com

Keeler BIO Face Shield Cleaning and Disinfection

Cleaning and disinfection should be done in accordance with the disinfectant manufacturer's indications for use; as well as OSHA, CDC, EPA and any applicable government or industry agencies.

Below are the recommendations from the CDC for reprocessing Face and Eye protection, outlined below, per <https://www.cdc.gov/coronavirus/2019-ncov/hcp/ppe-strategy/eye-protection.html>

1. While wearing gloves, carefully wipe the inside, followed by the outside of the face shield or goggles using a clean cloth saturated with neutral detergent solution or cleaner wipe.
2. Carefully wipe the outside of the face shield or goggles using a wipe or clean cloth saturated with EPA-registered hospital disinfectant solution.
3. Wipe the outside of face shield or goggles with clean water or alcohol to remove residue.
4. Fully dry (air dry or use clean absorbent towels).
5. Remove gloves and perform hand hygiene.

EPA COVID Approved Disinfectants:

<https://www.epa.gov/pesticide-registration/list-n-disinfectants-use-against-sars-cov-2>

CDC Disinfection Methods:

<https://www.cdc.gov/infectioncontrol/guidelines/disinfection/disinfection-methods/chemical.html>

Important Notes:

- This face shield is not an approved medical device nor a personal protective equipment under Keeler Ltd.
- This face shield is intended to only provide form of protection and minimize airborne infection transmission. It is not guaranteed to prevent airborne infection transmissions such as but not limited to coronavirus.

For more information please call 610.353.4350 or email customerservice@keelerusa.com
