


# Keeler

## TonoCare 2.0


### Three step guide

1. Position the forehead rest on the patient's forehead.
2. Look through the viewfinder and align the positioning circle concentrically to the patient's iris.
3. Adjust the forehead rest until the X fits the box and the TonoCare will do the rest.

POSITIONING  
CIRCLE

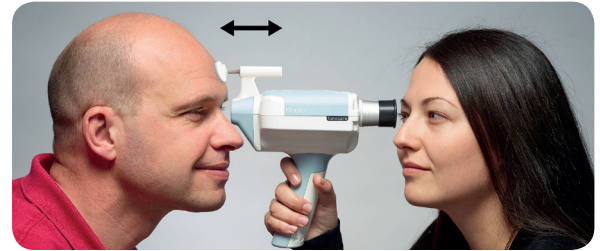

Centre  
on pupil

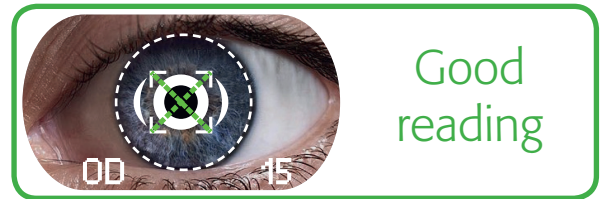

Good  
reading



Bad  
reading

ALSO CHECK OUT THE TONOCARE TRAINING VIDEO ON THE KEELER WEBSITE

# TonoCare 2.0

## Quick Start



Charge your TonoCare for a minimum of 12 hours before using it for the first time.



Perform a monthly 12 hour charge of your TonoCare to ensure optimum battery life.



Refer to the IFU for full instructions on how to use your TonoCare.

### Navigating the User Menu

- With the unit switched on, press and hold the menu button to enter the user menu.
- Scroll through the options using the print and return buttons.
- Short press the menu button to select an option and change it using the print or return buttons.
- Short press the menu button to return to the user menu.
- Press and hold the menu button to return to the measurement screen.

### Device Controls



On/Standby button



Menu button



Return button



Manual trigger



OD/OS button



Print button

### Sounds



Bad or invalid readings



Valid set of measurements obtained

### User Menu



Light



Sound



Time



Date



IOP Format



Pachymetry



Brightness



Self Test

### Description

Turn on/off illumination LEDs

Turn on/off audible alerts

Set the system time

Set the system date

Select the IOP format (XX/XX.X)

Pachymetry options (OFF/ON/TRIGGER)  
(If set to 'TRIGGER', the pachymetry option only appears if the measured IOP value is greater than 15)

Set the brightness for the displays  
(value between 1 and 15)

Initiate a self-test of the system

# Keeler

www.keeler.co.uk  
www.keelerusa.com