





D PAD

Back

Home

Blue Light  
Status Indicator

Trigger

Volume

Battery Cover

# Connecting VisuAll Headset to WIFI

---


## Step 1:

Hold and press the home button to connect remote to headset.

Then double press the home button quickly



Step 2:  
Point to WiFi  
icon and pull  
the trigger

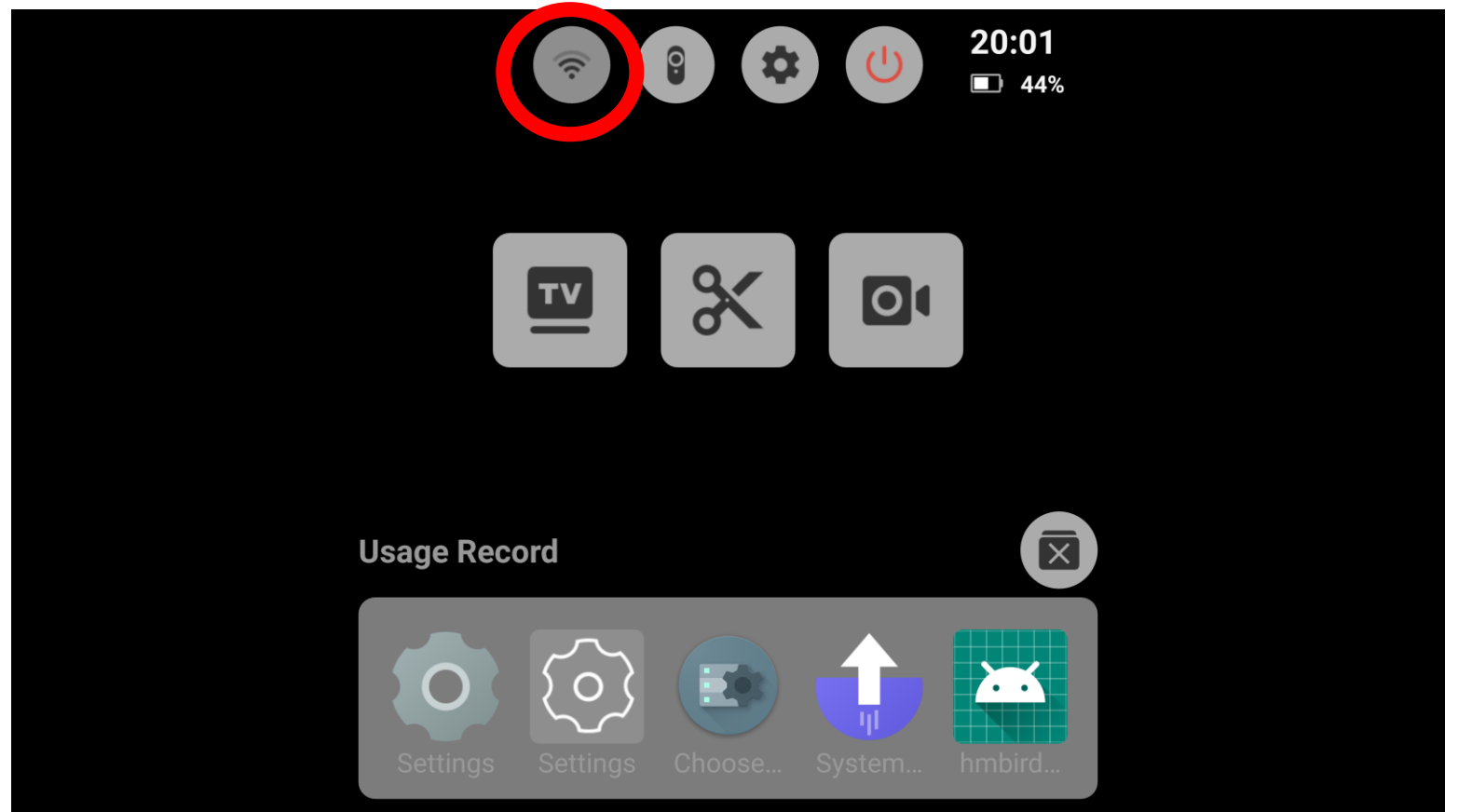


Step 3:  
Identify your  
Wifi  
connection.  
Point to it  
and pull the  
trigger



## Step 4:

Use the virtual keyboard to enter your password.

BEFORE clicking connect, please point and hit the ENTER button highlighted in the STAR.

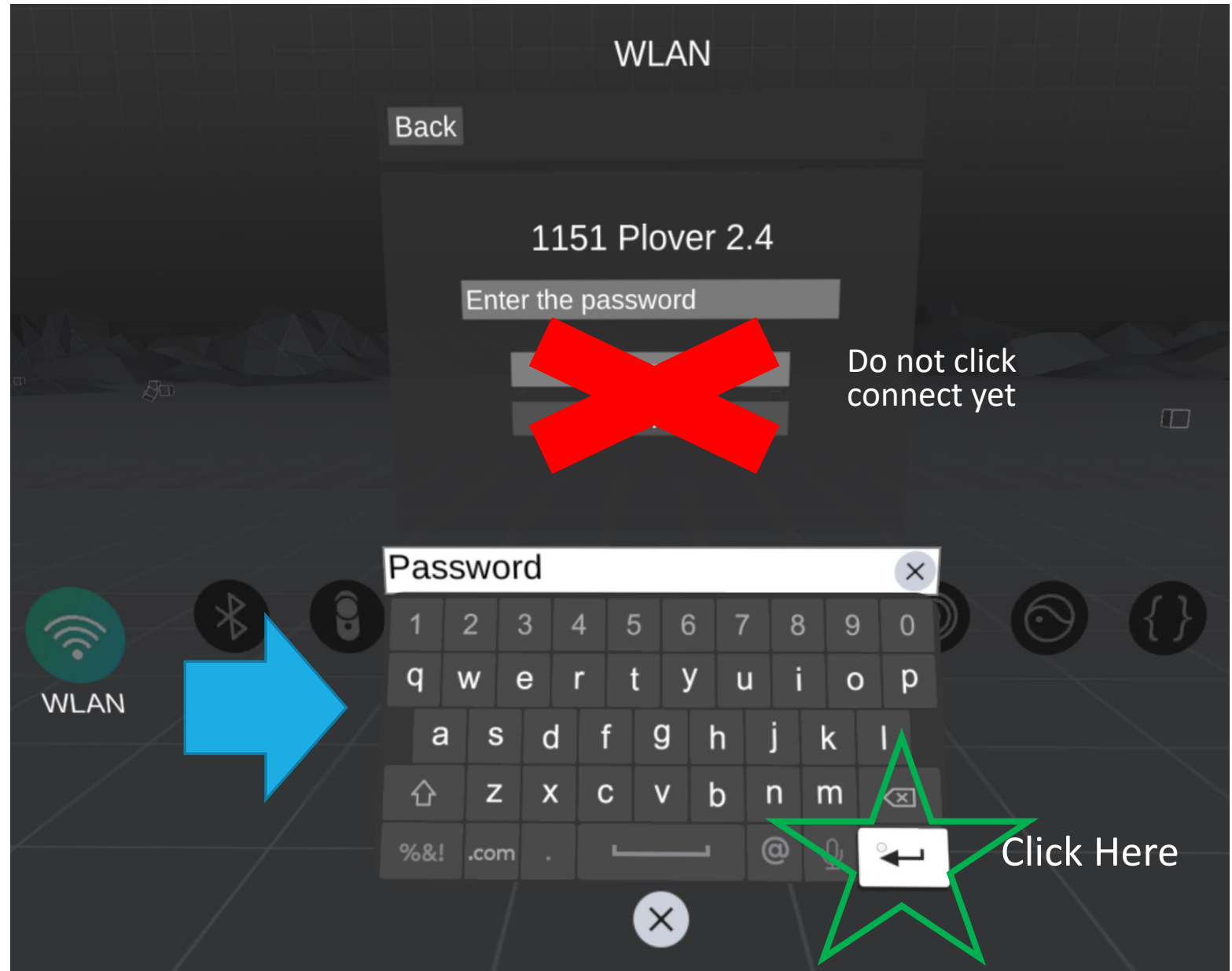

## Step 5:

After you enter your password and hit ENTER, Now point and click to CONNECT





## Step 6:

After a successful connection, a GREEN ✓ will appear next to the network name.

