


# Keeler Tonometer prism cleaning and disinfection instructions


Always disinfect the Tonometer prisms before use  
Hand hygiene must be considered to prevent any contamination

1

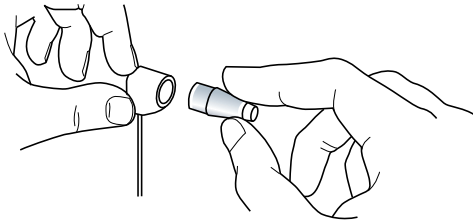


Carefully remove the Tonometer prism from the prism holder.

2

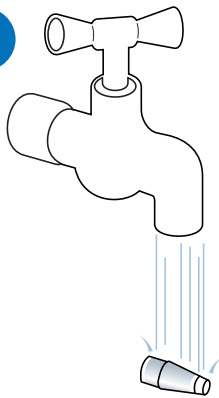

Wash the Tonometer prism under cold running water for approximately 1 minute, to ensure it is physically clean before exposed to disinfection process.


3

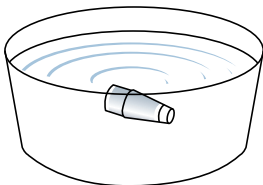

Immerse the Tonometer prism in the disinfectant fluid. Types of disinfectant fluid vary.


*Please follow disinfectant solution guidelines for instructions, concentration and time of immersion. (For example: Pantasept - 3% aqueous solution for 10 minutes, Hydrogen Peroxide 3% aqueous solution for 10 minutes, Sodium Hypochlorite, 10% aqueous solution for 10 minutes etc.).*

4

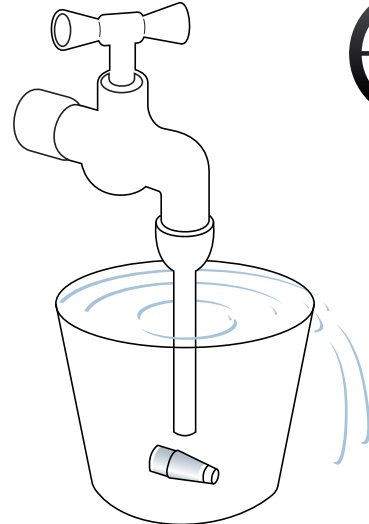


Rinse the disinfectant from the prism in running water for between 10 and 30 minutes.

5

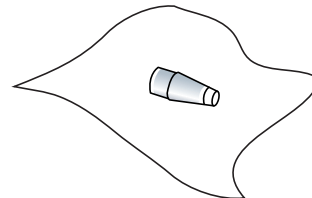

Dry the disinfected Tonometer prism with a clean soft cloth.

6

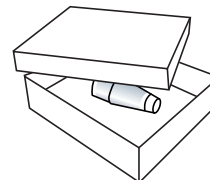

Store the Tonometer prism in a suitable container ready for use.

Safely dispose of the disinfectant fluids used.



**Do not disinfect using the following:** Alcohol, Acetone, UV radiation, Sterilisation, Immersion in fluid for more than one hour, Temperatures greater than 60°C

**Keeler**