

VisuALL S

Disinfection Guide



All VisuALL S surfaces must be disinfected prior to each use.

CAUTION: 1) The headset must not be charging during disinfecting process.
2) Do not spray the headset and/or handpiece with any liquid.

NOTE: The VisuALL S needs not be turned off during disinfecting process.

Instructions require alcohol wipes. Optional: Lens cleaning wipe.

Utilizing an alcohol wipe, thoroughly wipe the exterior body of the headset, the straps and the back rest.



Utilizing a new alcohol wipe or lens cleaning wipe, gently wipe each lens. While the lenses are not easily susceptible to scratches and damage, care should be taken to prevent abrasion.



Utilizing an alcohol wipe, thoroughly wipe the entire handpiece.

